Omnimed® Ortho Clavicula Universal

Rucksackverband universal

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Universal figure-of-eight strapping

Redressing bandage for treatment of clavicle fracture

Materials:

Self-adhesive polyester, polyester terrycloth, polyurethane foam, Velcro® hook & loop fastener, plastic elements with rotatable buckles

Application (only possible with outside help):

- Place the pad centrally between the shoulder blades and the straps fixed over the shoulders
- Guide the straps under the armpits towards the back
- Guide through the 2 buckles and determine the length of the straps by pulling. The closing pressure required for stabilisation builds up through tightening the straps.
- Cut the straps on the marked spots corresponding to the size.
- Fasten the 2 y-shaped Velcro units to the fitted straps.
- Guide the straps through the buckles once again. Make sure there is adequate tension in the opposite direction, and fasten the Velcro hook & loop fasteners on the back on the ends of the straps running between the armpits and pad
- The comfort padding can be shifted on the straps and positioned in the armpits.

Precautionary measures:

Venous congestions in the arms are to be avoided. The closing pressure of the bandage has to be adequately adjusted for this purpose. The correct fit should be double-checked by specialists and adjusted if necessary.

The figure-of-eight strapping is only intended for care of a patient and should not be used for other individuals due to hygienic reasons.

Store in a dry place!

Grössen / Dimensions / Taglie / Sizes:

Art. No.:
SX24500 für Erwachsene / pour adultes
per adulti / for adults
SX26400 für Kinder / pour enfants
per bambini / for children

SMEDICO AG
Rorschacher Strasse 304
CH 9016 St.Gallen, Switzerland
Tel (+41) 071 282 12 12
Fax (+41) 071 282 12 10

OMNIMED® is a registered trademark of
SMEDICO AG St.Gallen, Switzerland