GELLHORN

Multiple Drain Pessaries

Support for third-degree prolapse/procedentia
(Stage III or Stage IV)

Rx Only

AVAILABLE MODELS

GELLHORN, FLEXIBLE GELLHORN, 95% RIGID

GELLHORN, SHORT STEM

Single-Patient Use **€ 0086**

Wash pessary with mild soap and thoroughly rinse prior to initial use.

DESCRIPTION

The Milex® GELLHORN Pessary is a medical device made from silicone that is inserted into the vagina to function as a supportive structure of the uterus, bladder or rectum.

WARNINGS

- Do not leave pessaries in place for long periods of time, as serious complications can occur which may require surgical intervention. Do not use these pessaries on a patient with a known silicone allergy.

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CooperSurgical

PATIENT INSTRUCTIONS FOR PESSARY USE

CAUTION

COOPERSURGICAL RECOMMENDS PESSARIES BE INSERTED AND REMOVED BY THE PHYSICIAN OR OTHER HEALTHCARE PROFESSIONAL UNLESS OTHERWISE DIRECTED.

INDICATIONS FOR USE

- Chemicals in various vaginal preparations can interact with the pessary material, resulting in discoloration or deterioration of the pessary. TRIMO-SAN™ does not interact with the pessary material. Use only TRIMO-SAN. Other materials have not been tested for compatibility.
- For effective support of third-degree prolapse or procidentia. The cervix rests behind the flat base of the pessary and only the stem shows in the vaginal entrance when the patient does a Valsalva maneuver. The GELLHORN pessary requires a relatively capacious vagina and an intact perineum.

CONTRAINdications

- The presence of pelvic infections and/or lacerations
- A noncompliant patient
- Endometriosis
- Pregnant patient
- Sexually active patient

RECOMMENDED CLEANING INSTRUCTIONS*

- Prepare a cleaning solution by mixing a mild soap (such as Dawn® or equivalent) with tap water using the soap manufacturer's recommended concentration. Prepare this solution in a container large enough to fully submerge the device.
- Soak and Scrub
 - Soak the device in the container of prepared soap solution for a minimum of 5 minutes.
 - Following the 5-minute soak period, scrub the device for a minimum of 15 seconds with a soft-bristled brush, such as a tooth brush and/or pipe brush. Scrub device below water line to prevent aerosolization of contaminants.
 - Following scrub, inspect device for visible soil residue.

For medical emergencies and for all medically related advice, consult your healthcare professional.

FOR THE PHYSICIAN / HEALTHCARE PROFESSIONAL

Review these instructions with the patient to establish use regimen.

INSTRUCTIONS

- Wear dry gloves. When necessary, lubricate only the entering end of the pessary with TRIMO-SAN. Hold the pessary as shown in Figure 1.
- Use one finger to depress the perineum.
- Guide the pessary, inserting it edgewise almost parallel to the introitus (see Figure 1), avoiding the urethral opening while the perineum is strongly pushed downward. Use a corkscrew motion while introducing the GELLHORN into the vagina.

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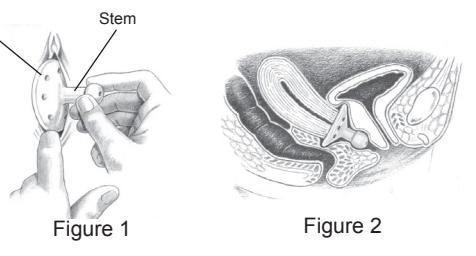
Figure 1

Figure 2

- During each visit, the vagina should be carefully inspected for evidence of pressure or allergic reaction. The patient should be questioned concerning douching, discharge, disturbance of bowel function or urination. It may be necessary to fit another size or an entirely different type of pessary.

TO REMOVE

- Use one finger to depress the perineum.
- Use other hand to grasp the "knob," pulling the pessary away from the cervix, turning the pessary so that the disc is almost parallel to the introitus. Using corkscrew motion ease the pessary out. See Figure 3.

RECOMMENDED FOLLOW-UP

- Have patient return within 24 hours for first examination.
- Have patient return for second examination within 3 days.
- Have patient return for examination every few months.

Note: The above schedule of follow-up examinations may be altered to fit the needs of the individual patient at the discretion of the healthcare provider.

RECOMMENDED FOLLOW-UP

- Have patient return within 24 hours for first examination.
- Have patient return for second examination within 3 days.
- Have patient return for examination every few months.

Note: The above schedule of follow-up examinations may be altered to fit the needs of the individual patient at the discretion of the healthcare provider.

RECOMMENDED FOLLOW-UP

- Have patient return within 24 hours for first examination.
- Have patient return for second examination within 3 days.
- Have patient return for examination every few months.

Note: The above schedule of follow-up examinations may be altered to fit the needs of the individual patient at the discretion of the healthcare provider.

RECOMMENDED FOLLOW-UP

- Have patient return within 24 hours for first examination.
- Have patient return for second examination within 3 days.
- Have patient return for examination every