English

Inflatoball™

Latex Rubber
(100% Latex)

MILEX™

INFLATOBALL™ Latex Rubber Pessary

For a mild cystocele or rectocele associated with proctodia/prolapse

Fitting Instructions for the Health Care Professional

Patient instructions enclosed

Single-Patient Use

Wash Pessary with mild soap and thoroughly rinse prior to initial use.

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.

CooperSurgical

Made in the USA

95 Corporate Drive
Trumbull, CT 06611 USA
Phone: (800) 243-2974
Fax: (800) 262-0105

International
Phone: (203) 601-9818
Fax: (203) 601-4747

EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands

www.coopersurgical.com

37804 • Rev. A • 9/12

DESCRIPTION

The INFLATOBALL™ pessary is a single-patient use natural latex pessary for mild cystocele and/or rectocele associated with proctodia/prolapse.

AVAILABLE MODELS INFLATOBALL

NOTE: The patient must be taught how to insert, remove, and clean this pessary nightly.

[REF] MXKPINF + Size

To properly fit a patient with an INFLATOBALL pessary, it is necessary to have available a minimum of two sizes. The shaded area indicates sizes recommended for office fitting. With these sizes you will be able to fit approximately 85% of patients.

AVAILABLE IN THE FOLLOWING SIZES

Inflated Sizes	Diameter in inches	Diameter in mm
Small (S)	2 ± 0.050	51 ± 1.27
Medium (M)	2-1/4 ± 0.050	57 ± 1.27
Large (L)	2-1/2 ± 0.050	64 ± 1.27
Extra Large (XL)	2-3/4 ± 0.050	70 ± 1.27

CPT CODE

Procedure: 57160

HCPCS: A4561

IMPORTANT

The INFLATOBALL (latex rubber) pessary should NOT be used in a patient using ANY vaginal hormone cream or suppository including Premarin® Cream or suppository.

Vaginal hormonal products contain a wax base which will deteriorate latex rubber.

Medical-grade silicone pessaries will not be adversely affected by wax-based medications.

Use only TRIMO-SAN™; other materials have not been tested for compatibility.

DO NOT OVER INFLATE.

Overinflation distorts and weakens latex. If your pessary “balloons” out, it is an indication that it has been overinflated and you need a larger size INFLATOBALL or a different type of pessary.

No credit or replacement for the pessary will be allowed because of overinflation or if used contrary to the above restrictions.

Review the separate instructions with the patient to establish use regimen.

FITTING INSTRUCTIONS FOR THE HEALTH CARE PROFESSIONAL

IMPORTANT

- Pessaries are fitted by trial and error. There are no mechanical devices available that can accurately determine the size or type of pessary your patient requires to obtain the desired results.
- Fitting diaphragms **should not** be used to measure the size pessary a patient will need. Diaphragms fit differently from pessaries.
- Even before fitting a pessary, the patient should be informed that it is not uncommon to have to change the size or type of pessary more than once after being originally fitted. This is why it is so important that your patient be instructed to return within 24 hours of the initial fitting and again in 72 hours. Thereafter, re-examination every few months is recommended to ensure a proper fit is maintained as long as the patient is wearing the pessary.
- At each visit the pessary should be removed and the vaginal vault inspected for signs of allergic reaction or undue pressure.
- At the physician's discretion, the patient can be instructed in the proper removal, cleaning and reinsertion techniques for her own pessary. This process can be performed nightly by the patient under ideal circumstances.
- The INFLATOBALL, CUBE and TANDEM-CUBE pessaries are the three exceptions to the above. Patients must be instructed to insert, remove and clean these pessaries daily.
- The CUBE and TANDEM-CUBE pessaries fill the entire vaginal vault and have no area for drainage. The INFLATOBALL pessary is made of latex rubber and should not be left in place more than 24 consecutive hours.
- A noncompliant patient should not be fitted with any pessary. It is essential that your patient understands the importance of these frequent follow-up visits and that she fully cooperates with you to ensure the desired results.

Note: Each type of pessary is available in a wide range of sizes – most pessaries are made in 9 to 14 different sizes.

Experience has shown that in order to properly fit an INFLATOBALL pessary, you should have at least one of the two most commonly used sizes of this pessary (sizes medium and large).

INDICATIONS:

For effective support in second- or third-degree prolapse. The INFLATOBALL is also effective in patients with a mild cystocele or rectocele associated with a proctodia/prolapse. Because the support can be adjusted based on the inflation level of the pessary, the INFLATOBALL is sometimes the only pessary the patient can tolerate.

WARNINGS

The MILEX™ INFLATOBALL pessary is made of latex rubber.

As a result of a MEDICAL ALERT issued by the FDA, March 29, 1991 on allergic reactions to latex-containing medical devices, the following recommendations are made:

- Question the patient on latex sensitivity (itching, rash, wheezing, when wearing latex gloves or inflating a balloon).
- Establish a regimen of frequent re-examinations after the initial 24- and 72-hour rechecks following the pessary fitting.
- All patients wearing an INFLATOBALL pessary should be re-examined every 4 to 6 weeks. At these times, the vagina should be inspected for signs of allergic reaction or evidence of pressure. Question the patient concerning itching, rash, wheezing, as well as symptoms such as discharge, and disturbance of bowel and urinary function.
- Instruct the patient on how to remove the INFLATOBALL pessary and report to you immediately if symptoms of itching, rash, and/or wheezing occurs. Pessaries of alternate materials are available.

CONTRAINDICATIONS:

- Sensitivity or allergy to latex rubber
- The presence of pelvic infections or lacerations
- A noncompliant patient
- Endometriosis has been suggested as a possible contraindication to pessary use
- Health care professionals should use their own professional judgment as to the advisability of using this pessary in a pregnant patient

Note: Since the INFLATOBALL pessary is manufactured of a soft latex rubber, it will absorb mucus and vaginal secretions and should be removed nightly and cleaned.

PESSARY CARE:

DO NOT AUTOCLAVE OR BOIL.

Pessaries are powdered with food-grade powder. The powder must be washed off with a mild soap and the pessary thoroughly rinsed with water and dried prior to initial use.

DISINFECTION / DECONTAMINATION¹:

The "bead" should be at the bottom (closed end) of the stem. Remove the air from the INFLATOBALL pessary. Keeping the pessary compressed, move the "bead" approximately 1/2 inch to 3/4 inch up the stem (away from the closed end) to close off the stem so the solution will not enter the pessary. Do not submerge the bulb in water or any solution.

Wash the pessary with mild soap and water. Rinse thoroughly. Immerse in 70% isopropyl alcohol or 80% ethanol alcohol for 20 minutes. Rinse thoroughly.

Note: the INFLATOBALL pessary should be periodically checked for deep cracks or scratches that could make adequate cleaning and disinfecting impossible.

¹Lettau, L.A., M.D., MPH. Centers for Disease Control: JAMA, August 9, 1985, page 752.

Reprint available upon request.

PRIOR TO FITTING:

Have the patient empty her bladder before fitting this pessary.

Ulcerations and erosions frequently occur in cases of complete prolapse due to irritation of the exteriorized cervix.

Whenever possible, reducing the mass and treating the irritation are primary steps before using a pessary.

The only method of determining the proper size INFLATOBALL pessary is trial and error. Because latex rubber will absorb secretions and odors, this pessary should be removed nightly and cleaned. The INFLATOBALL pessary is designed for ease of insertion and removal by the patient. The pessary is inserted DEFLATED.

INSTRUCTIONS FOR USE

STEP-BY-STEP FITTING INSTRUCTIONS FOR THIS PESSARY

In general, the largest pessary that a patient can comfortably accommodate should be fitted.

Perform a normal pelvic examination before inserting or fitting of any pessary. A first approximation of size can be made by using your fingers to determine the approximate width of the vaginal vault. This will generally get you within a size or two of the proper pessary.

Note: If necessary, irrigate the vagina prior to insertion of the pessary. This will cleanse the vagina of excess discharge and secretions. HCPCS Supply Number A4320 (Irrigation Tray with Syringe, any purpose).

1. Deflate the pessary by moving the "bead" to the closed end of the stem (see Figure 1). Apply light pressure with the thumb and fingers to force additional air from the pessary (see Figure 2).

Figure 1

Figure 2

2. When the pessary is deflated and the "bead" is at the bottom of the closed end of the stem, insert the metal end of the bulb into the open end of the stem (see Figure 3).

Figure 3

3. Hold the deflated pessary between the thumb and fingers for insertion. If necessary, the leading edge of the INFLATOBALL may be lubricated with TRIMO-SAN vaginal jelly or other water-based gel.

Figure 4

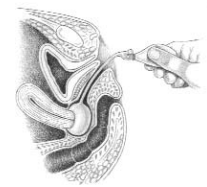
Figure 5

4. Insert the pessary into the vagina so that only the stem protrudes (see Figure 4).
5. Inflate the pessary by squeezing the bulb. The inflation level can be controlled by the number of pumps or squeezes on the bulb (see Figure 5). **DO NOT** overinflate the pessary.

6. Push the small "bead" (located at the tip of the stem) about 3/4 inch to 1 inch forward. This will close the air vent and will keep the pessary inflated while it is being worn (see Figure 6).

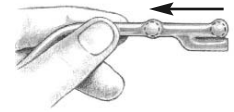
Figure 6

7. Remove the bulb end from the air stem. The stem can then be tucked into the vaginal vault.
8. Have the patient sit, stand and bear down. Examine the patient while she is in the standing position to ensure the pessary has not shifted position. The patient should not feel the pessary once it is in position. The pessary should not be too loose as it may turn or be expelled, and it should not be too tight as it may cause discomfort.
9. The health care professional should be able to sweep one finger between the pessary and vaginal walls. If there is not enough space to do this, the next smaller pessary should be tried. If excessive space exists, the pessary will not be effective and may rotate or even be expelled.
10. It is sometimes necessary to refit the patient with a different size or type of pessary after a period of time. Do not assume that a replacement will always be the same size as the previous one. Check the fitting to ensure continued patient comfort and relief of symptoms. The use life of a pessary is limited. Examine frequently for signs of deterioration.

PATIENT FOLLOW-UP:

Have the patient:

- Report immediately any difficulty in urinating
- Report immediately any discomfort
- Return within 24 hours for first examination
- Return for second examination within 3 days
- Return for examination every few months

Note: Above schedule of follow-up examinations may be altered to fit the needs of the individual patient.

TO REMOVE:

To remove the pessary, push the "bead" down to the closed end of the stem to permit air to escape from the pessary (see Figure 1). Now grasp the deflated pessary and withdraw. **DO NOT** pull on the pessary stem as this can be more traumatic to the patient.

During each visit, the vagina should be carefully inspected for evidence of pressure or allergic reaction. The patient should be questioned concerning discharge, disturbance of bowel function or urination. It may be necessary to fit another size or an entirely different type of pessary.

DO NOT assume that a replacement will be the same size as the previous one. Check the fitting to ensure patient comfort and relief of symptoms.

At each checkup, the pessary is removed and cleaned. If there are no contraindications, the pessary is reinserted.

WARNING: Chemicals in various vaginal preparations can interact with the pessary material, resulting in discoloration or deterioration of the pessary. TRIMO-SAN does not interact with the pessary material. Use only TRIMO-SAN; other materials have not been tested for compatibility.

PURPOSE OF TRIMO-SAN:

1. To help restore and maintain normal vaginal acidity.
2. To coat the walls of the vagina with a lubricating film that helps reduce odor-causing bacteria.

APPLICATION:

- Pessary wearers should use 1/2 applicator of TRIMO-SAN three (3) times the first week after original insertion of the pessary, unless otherwise directed by the health care professional.

- Use 1/2 applicator of TRIMO-SAN twice a week thereafter, unless otherwise directed by the health care professional.
- Reaction or irritation caused by TRIMO-SAN is very rare, but should it occur, discontinue use.
- If symptoms persist or worsen, contact your health care professional.

TRIMO-SAN HAS NOT BEEN TESTED DURING PREGNANCY:

- If the patient wishes to become pregnant, please advise her against using TRIMO-SAN or any other personal hygiene product (douches, vaginal sprays, jellies or suppositories) for 6 hours prior to or following intercourse. Any product used within this time period could interfere with conception.

SUGGESTION: Advise your patient that TRIMO-SAN is available at the prescription counter upon request (no Rx required).

(SEE PACKAGE INSERT FOR TRIMO-SAN USE COMPLETE DIRECTIONS)

HAVE PATIENT REPORT ANY OF THE FOLLOWING SYMPTOMS:

- Any changes in the color or consistency of vaginal discharge
- Any increase in amount of vaginal discharge
- Any foul odor associated with vaginal discharge
- Vaginal itching

For medical emergencies and for all other medically-related advice, consult your health care professional.

Review separate instructions with the patient to establish use regimen.

EXPLANATION OF SYMBOLS

REF Catalog Number

LOT Batch Code

 Use By (year, month)

 **ATTENTION:**
See instructions for use

Rx ONLY **CAUTION:** U.S. Federal law restricts this device to sale by or on the order of a physician

EC REP Authorized Representative in the European Community.

 **0086** Product conforms to the Medical Device Directive 93/42/EEC.

Premarin® is a registered trademark of Wyeth LLC.

Milex™, TRIMO-SAN™ and INFLATOBALL™ are trademarks of CooperSurgical, Inc.

CooperSurgical is a registered trademark of CooperSurgical, Inc.

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© 2012 CooperSurgical, Inc.