SLINGSHOT 2™ SLINGSHOT NEUTRAL™ KOOLSLING™

FITTING INSTRUCTIONS

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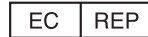
MISE EN PLACE

INSTRUCCIONES DE AJUSTE

Breg, Inc.
2885 Loker Ave. East
Carlsbad, CA 92010 U.S.A.
P: 800-321-0607
F: 800-329-2734
www.breg.com

E/U authorized representative
MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany

AW-1.90058 REV C 0213

⚠️ WARNINGS

CAUTION: FOR SINGLE PATIENT USE ONLY.

CAUTION: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

WARNING: CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE.

WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.

1 While supporting the injured arm, slide the forearm into the sling. Wrap the adjustable **shoulder strap** around the neck and shoulders and attach the strap end to the sling using the **"alligator strap tab."**

2 Adjust the **shoulder strap** to the required length.

3 Position the **shoulder pad** behind the neck.

To secure the **shoulder pad location** (optional), pull the shoulder strap away from the shoulder pad, unfold the **hook material** on the top of the pad, then press the shoulder strap back onto the hook material to secure the pad in place.

4 Attach the **closure strap** across the open top of the sling. Position the strap so that it holds the arm securely in the sling. Next, attach the **thumb strap** to the open end of the sling between the thumb and fingers.

5 After sling has been fit, it may be easily removed and reapplied using the **quick release buckle** on shoulder strap.

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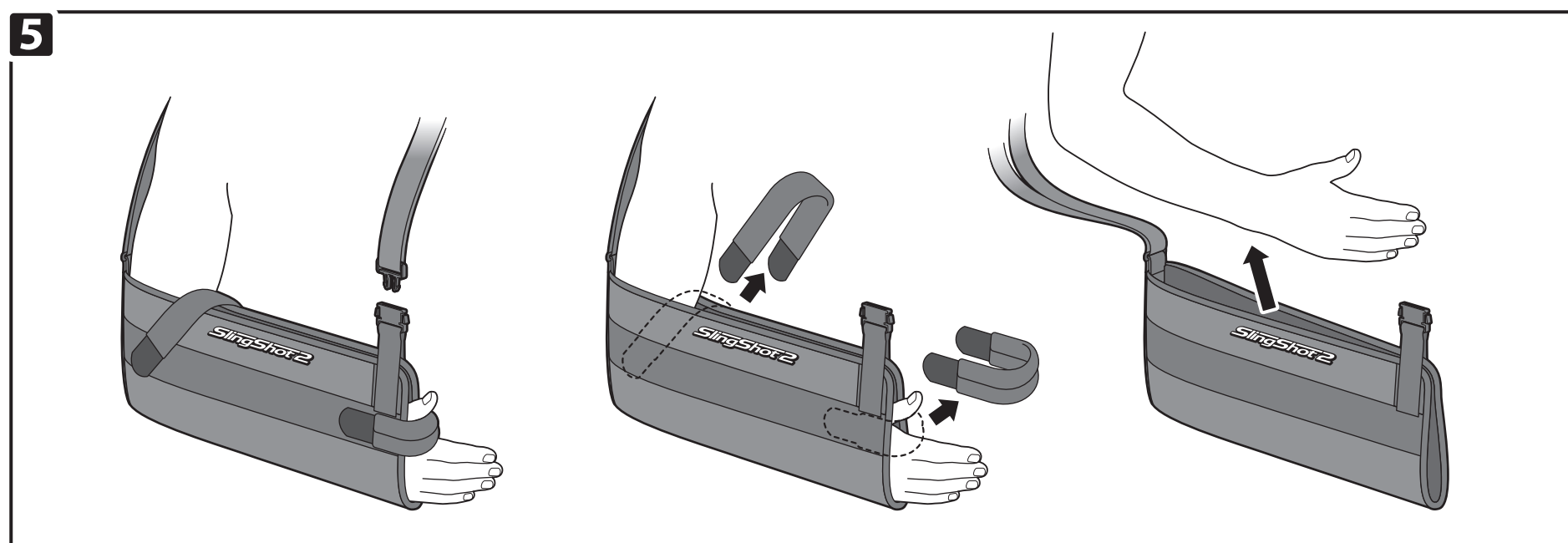
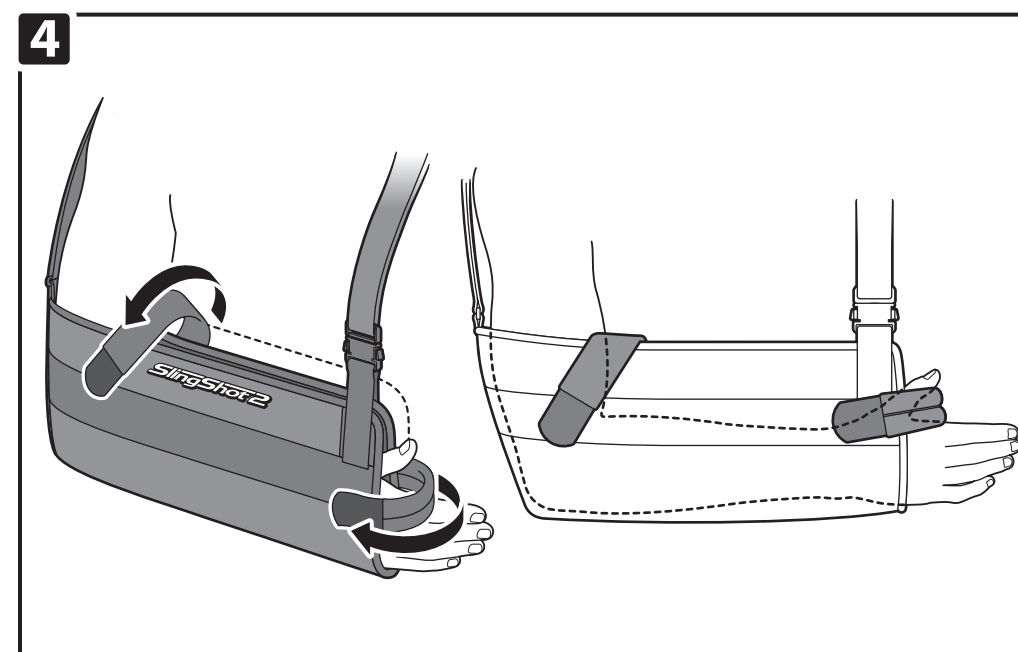
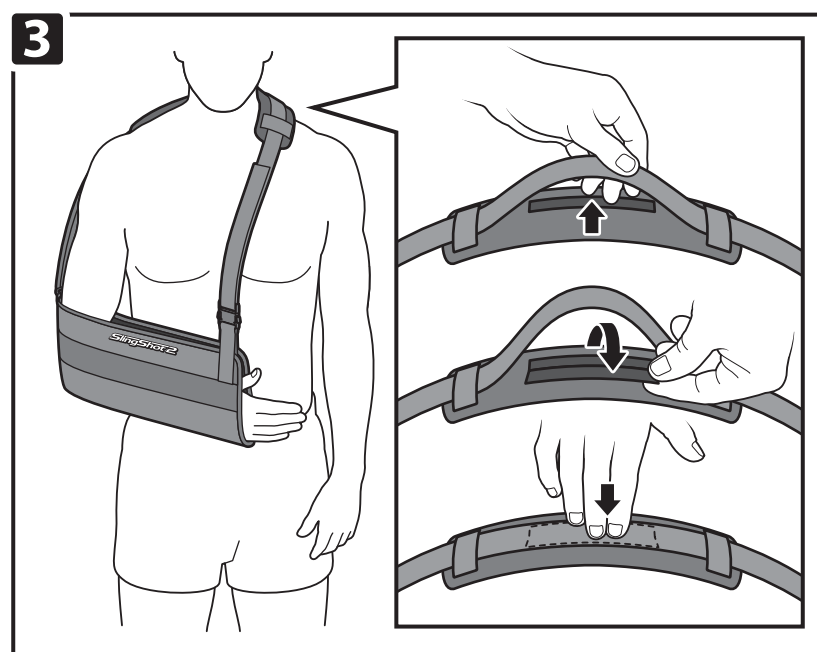
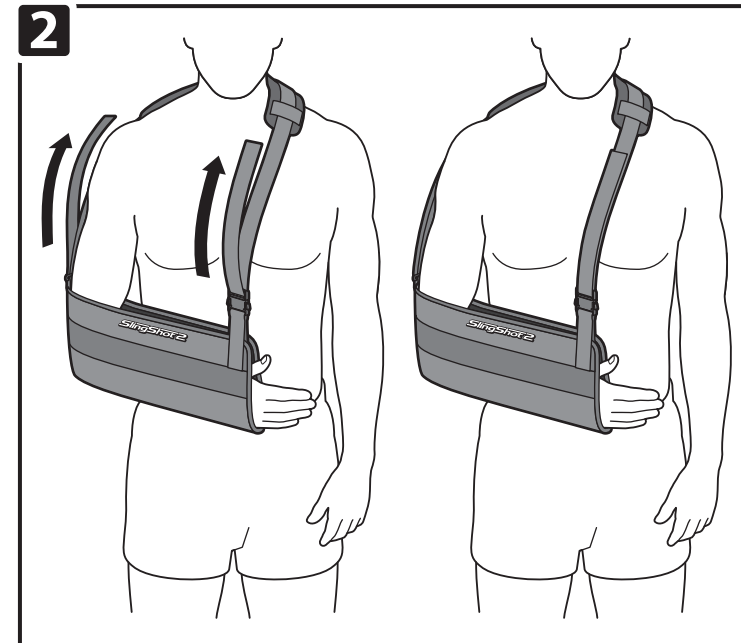
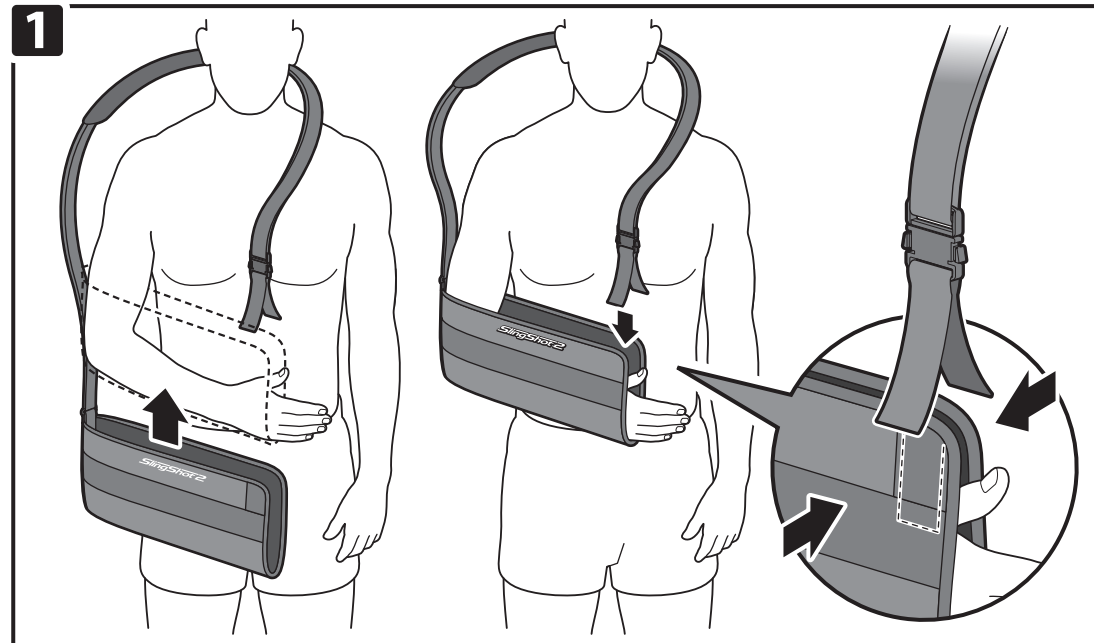
6 If a **neutral pillow** or **15° abduction pillow** is included, place the pillow at the waistline. Attach the sling to the pillow, lining up hook material on the pillow with the loop on sling. Adjust the waist strap to fit.

7 If waist strap is too long, cut it to fit. Use the small piece of **double sided hook material** (located on top of the pillow) to secure the strap end. Place the double sided hook material on the inside of the cut strap end and secure it to the waist strap.

8 If no pillow is included, attach the waist strap to the sling and adjust to fit.

Washing Instructions: Straps and sling must be removed and cleaned regularly depending on your activity level and perspiration. Hand wash straps and sling in cold water with mild detergent, rinse, air dry.

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